

Oticon ♦ Tego

Experienced users

## What to expect from Tego



oticon  
PEOPLE FIRST

# Meet your new personal assistant!

Now that you've taken the first steps towards upgrading to Tego, you're probably wondering what to expect.

Well, you can expect a lot! You have selected an instrument that continuously adjusts to fit your personal needs. Tego does this with unmatched precision, thanks to the unique DecisionMaker. This smart new system is based on Artificial Intelligence.

## Say goodbye to poor sound quality

Tego will be like your new personal assistant! Tego is so cleverly designed that to hear considerably better all you'll have to do is to insert it in the morning, take it out at night and change the batteries when Tego tells you to.

Read on to learn more about these unbeatable new features!



# Say hello to new benefits

DecisionMaker continuously combines several advanced sound processing systems, allowing “hands free” operation. All the while, the sound quality is exceptionally clear and comfortable. Here’s a little more on how it works.

## **Hear the speech ...**


Tego has a new directional system, which cleverly determines what you need to hear – be it from a voice in front or behind you. If there is no specific speaker, Tego keeps you comfortably connected to your surroundings. This system focuses on voices and yet keeps the overall sound spacious and natural.

## **... not the noise**

Things like car engines and dishwashers can be very distracting when all you want to do is talk! Tego’s noise reduction and directional systems actually work together to form a two-pronged attack on unwanted noise. Tego enhances the speech you want to hear AND provides a comfortable listening experience. With Tego, it’s not one or the other.

## **Freedom from feedback**

Everyone hates whistling sounds, not only because they’re loud but also because they occur at the most inopportune moments! Tego loves feedback though: finding it and eliminating it before it’s even heard.

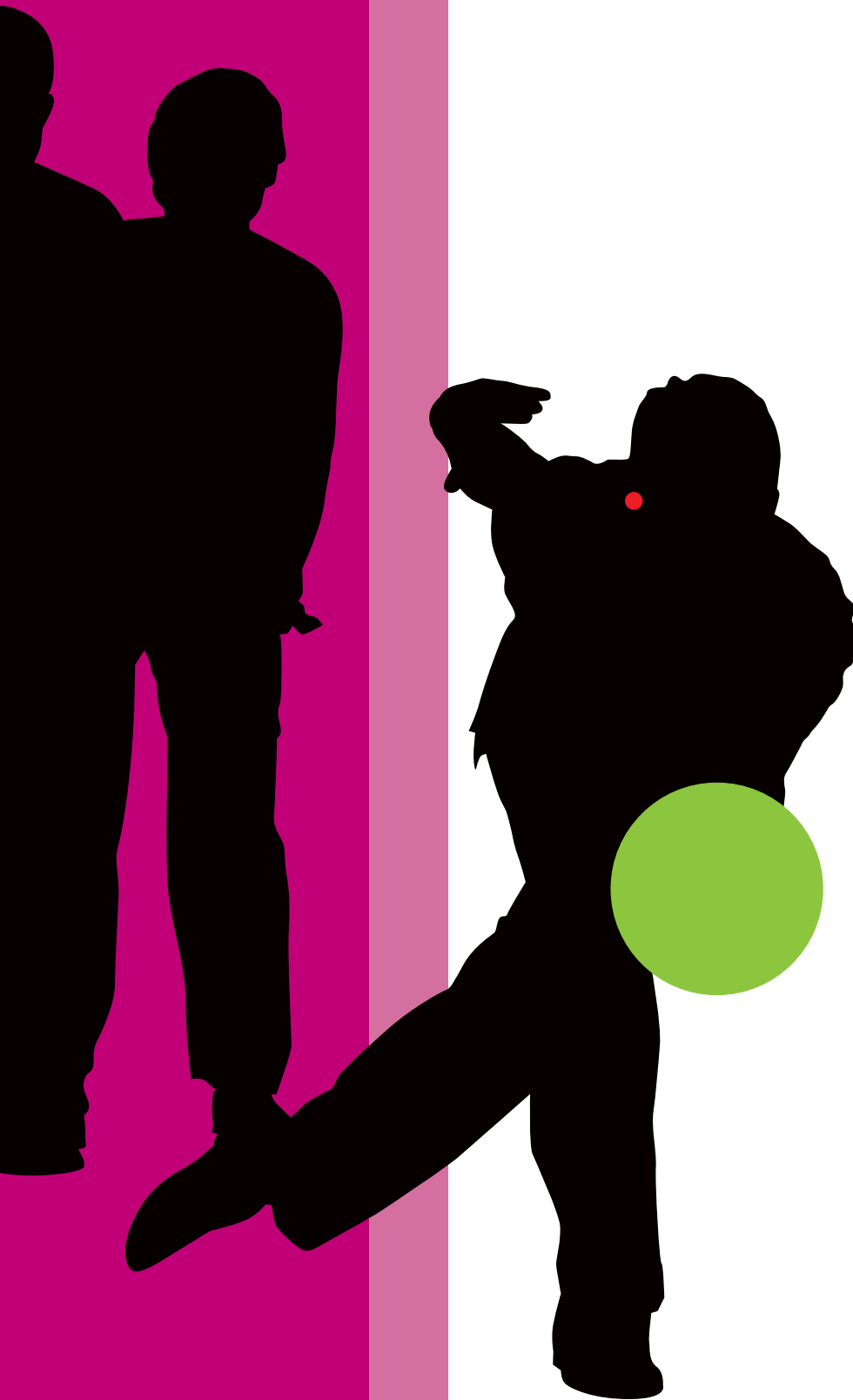

# Personal counselling

We will check that your instruments are comfortably fitted and that their performance matches your personal hearing profile and the kind of lifestyle you lead.

When you return for a follow-up visit, we will discuss how you're getting on with the instrument. We will review your impressions and your progress, and if Tego needs to be slightly adjusted, or you want the sound quality altered, we can make the changes easily.

## **Good luck with your personal assistant!**

You have taken the first steps towards upgrading your hearing with Tego. In just a few days you will notice a difference – and your friends and family will notice a difference – because they will see you becoming more confident, more active and more energetic every day.



# People first



We believe that it takes more than technology and audiology to create the best hearing instruments. That's why we put the individual needs and wishes of people with hearing loss first in our development of new hearing care solutions.

[www.tego.oticon.com](http://www.tego.oticon.com)  
[www.hear-it.org](http://www.hear-it.org)

**oticon**  
PEOPLE FIRST